

Monitor Outfitters Cuff Cleaning Instructions

CLEANING AND INSPECTION

- Prior to use, check that the cuff, tubing, and hose are clean and free of damage. Replace cuff when aging, tearing, or weak closure is apparent.
- Wipe or brush off any visible soil.
- Apply a mild enzymatic detergent (ENZOL, Cidezyme or equivalent) to a damp cloth and wipe clean with distilled water.
- Inspect the tubing and bladder for signs of degradation, cracks, holes, or damage after each cleaning.
- Do not allow fluid to enter the tube(s). This may affect pressure accuracy or damage equipment.

DISINFECTANT

- After cleaning, spray cuff with a 10% bleach solution until surface is saturated.
- Let stand for 2 minutes
- Wipe dry with a clean cloth
- Disinfectant may cause discoloration.